



stretcher scale operation

for use with the 5400 & 7500 Series Stretchers



STRETCHER SCALE

Backlit LCD scale display reads to the nearest pound with a +/- margin of 5 pounds throughout the entire 750 pound range. Instrumentation cable is routed through frame so as not to interfere with patient or peripheral components.

Part No.: 750214

Basic Operation Instructions for the Stretcher Scale:



Note: For proper scale operation the litter top cannot be resting on the center column support. Raise litter top from center column support by pumping hydraulic pedal once or twice.

ZERO



Zero: Resets the scale to zero out the weight. This only needs to be done when the stretcher is first set up or if any accessories or additional gear are added or removed from the litter top.

WEIGH



Weigh: Press this button to collect a weight.

RECALL / HOLD



Recall / Hold:

This button has two functions.

1. Press and release this button after you have weighed a patient and it will recall the last weight even after the patient is off the scale
2. With a patient on the scale you can press and hold this button to change from the standard hold mode to continuous mode.

Hold mode: This is the default mode. In hold mode when you press the Weigh button the scale weighs the patient and stores the weight to memory so the weight can be recalled later.

Continuous mode: In this mode you can see a real-time weight. Similar to that of a bathroom scale, the display will fluctuate as weight increases or decreases. In this mode the weight is not logged into memory. To log a weight in the continuous mode you have to press and hold the Recall / Hold button to capture the weight.

MODE



lb / kg: Changes the scale mode from pounds to kilograms.