



Table Care & Cleaning Instructions

Tables

Stainless Steel

- SG-80-SS
- SG-80-A-SS
- SG-80-B-SS
- SG-80-C-SS
- SG-81-SS
- SG-82-SS
- CDS-1000
- CDS-1100
- CDS-2472
- CDS-3260
- CDS-3684
- P-5182
- P-5182-E
- SG-83-SS
- SG-84-SS
- SG-85-SS
- SG-86-SS
- SG-87-SS
- SG-88-SS
- SG-89-SS
- SG-90-SS
- SG-91-SS
- SG-92-SS
- SG-93-SS
- SG-94-SS
- SG-95-SS
- SG-96-SS
- SG-97-SS
- SG-98-SS
- SG-99-SS

Caution

DO NOT CART WASH. Product may be damaged by exposure to the high temperatures.

Disinfection

Disinfection may be achieved by use of hospital grade chemical disinfectants. This includes up to 10% bleach solution or bleach wipes. Refer to the disinfectant manufacturer's instructions for use.

Equipment Required:

Disinfectant, water dampened cloth

Disinfectant Instructions:

1. Apply the disinfectant to the product per manufacturer's instructions.
2. Let the product stand for the exposure time per the manufacturer's instructions.
3. Wipe the product with a water dampened cloth to remove disinfectant residual.
(It is recommended that the product be wiped down using a water dampened cloth after disinfection to prevent possible corrosion of the product's materials caused by overexposure to the disinfectant.)

Cleaning

Cleaning is required only if the product is heavily soiled and standard disinfection is not sufficient. Be sure to follow the detergent manufacturer's instructions for use.

Equipment Required:

Detergent, soft bristle brush, lint free cloth, running water

Cleaning Instructions:

1. Rinse excess soil from all surfaces
2. Apply detergent per manufacturer's instructions. Apply to all surfaces and use the brush to scrub surfaces as required. (Brushing along the material grain will reduce the noticeability of any scratches.)
3. Thoroughly rinse all surfaces of residual detergent and soil. See manufacturer's instructions for details.
4. Dry with lint free cloth or per manufacturer's instructions.