



# Hamper Care & Cleaning Instructions

## Hampers

### Stainless Steel

- P-1120-L-SS
- P-1121-L-SS
- P-1120-SS
- P-1121-SS
- P-1130-L-SS
- P-1130-SS

### Powder Coated

- P-120-L
- P-121-L
- P-120
- P-121

## Caution

**DO NOT CART WASH.** Product may be damaged by exposure to the high temperatures.

## Disinfection

Disinfection may be achieved by use of hospital grade chemical disinfectants. This includes up to 10% bleach solution or bleach wipes. Refer to the disinfectant manufacturer's instructions for use.

### Equipment Required:

Disinfectant, water dampened cloth

### Disinfectant Instructions:

1. Apply the disinfectant to the product per manufacturer's instructions.
2. Let the product stand for the exposure time per the manufacturer's instructions.
3. Wipe the product with a water dampened cloth to remove disinfectant residual.  
(It is recommended that the product be wiped down using a water dampened cloth after disinfection to prevent possible corrosion of the product's materials caused by overexposure to the disinfectant.)

## Cleaning

Cleaning is required only if the product is heavily soiled and standard disinfection is not sufficient. Be sure to follow the detergent manufacturer's instructions for use.

### Equipment Required:

Detergent, soft bristle brush, lint free cloth, running water

### Cleaning Instructions:

1. Rinse excess soil from all surfaces
2. Apply detergent per manufacturer's instructions. Apply to all surfaces and use the brush to scrub surfaces as