

Blanket Warmer

P-2012

P-2022

P-2032

P-2210

P-2220

P-2230

P-2240



MN-39931

REV.04
8/25

EN

Manufacturer's Information

Copyright

© Copyright 11/20 by Pedigo Products, Inc.

All rights reserved.

This manual or any portion thereof may not be reproduced or used in any manner whatsoever without the express written permission of Pedigo Products, Inc.

Trademarks

All trademarks referenced in this documentation are the property of their respective owners.

Manufacturer

Pedigo Products, Inc.

4000 S.E. Columbia Way

Vancouver, WA 98661, USA

1-800-822-3501

www.pedigo-usa.com

Original instructions

The content in this manual is written in American English.



Thank you for your Purchase!

This warmer has been thoroughly tested and inspected to ensure only the highest quality is provided. We supply the most durable, convenient, efficient and safe warming equipment on the market. All warmers are manufactured and fully inspected in the USA with a commitment to quality.

Pedigo Repair Service

Call

Call 800-822-3501 to reach our service call center for access to local authorized service agencies. This service is provided exclusively for Pedigo Product, Inc. equipment and is available throughout the United States. Our call center is available M-F 7:00 a.m. to 5:00 p.m. PST.

This page intentionally left blank.

Manufacturer's Information	2
Foreword	3
Thank you for your Purchase!	3
Pedigo Repair Service	3
Safety	7
The Meaning of Signal Words	7
Safety Precautions.	8
Labels	11
Label Locations	11
Specifications	13
Specification Information	13
Dimension Drawings	14
Installation	29
How to Unpack the Warmer	29
How to Install the Warmer	30
Operation	33
Preparing the Warmer for First Use.	33
How to Turn On and Turn Off the Warmer.	34
How to Operate the Blanket Warmer	35
How to Change the Temperature Scale.	36
How to Change the Sound Settings	36
How to Lock and Unlock the Controller.	37
How to Adjust the Interior Light (if equipped)	38
Maintenance	39
Maintenance Schedule	39
How to Clean the Warmer	41
Warmer Disposal / Decommissioning	42
Troubleshooting	43
What to do if a Power Interruption Occurs.	43
What to do if the Alarm Indicator Light Flashes	44
Error Codes.	45
How to Replace a Fuse	49
Electrical Information	51
Warranty	55
Warranty	55

This page intentionally left blank.

The Meaning of Signal Words

This manual contains signal words where needed. These signal words must be obeyed to reduce the risk of death, personal injury, or equipment damage. The meaning of these signal words is explained below.

**DANGER**

Danger indicates a hazardous situation which, if not avoided, will result in serious injury or death.

**WARNING**

Warning indicates a hazardous situation which, if not avoided, could result in serious injury or death.

**CAUTION**

Caution indicates a hazardous situation which, if not avoided, could result in minor or moderate injury.

NOTICE

Notice indicates a situation which, if not avoided, could result in property damage.



NOTE: Note indicates additional information that is important to a concept or procedure.

Safety Precautions

Before you begin

Read and understand all instructions in this manual.

Electrical precautions

Obey these electrical precautions when using the warmer:

- Connect the warmer to a properly grounded outlet. Do not use the warmer if it is not properly grounded. Consult an electrician if there is any doubt that the outlet used is properly grounded.
 - Keep the cord away from hot surfaces.
 - Do not attempt to service the warmer or its cord and plug, when plugged in.
 - Do not operate the warmer if it has a damaged cord or plug.
 - Do not immerse the cord or plug in water.
 - Do not let the cord hang over the edge of a table or counter.
 - Do not use an extension cord.
-

Usage precautions

Obey these usage precautions when using the warmer:

- Only use this warmer for its intended use of warming medical solution bags, bottles, and/or blankets. Follow facility and solution manufacturer guidance regarding warming temperature for any item to be placed in the warmer.
 - Do not use this warmer for warming blood or blood products.
 - Do not cover or block any of the openings of this warmer.
 - Do not use this warmer in a wet location.
 - Only clean the warmer when the power cord is unplugged.
 - Do not use corrosive chemicals when cleaning the warmer.
 - Do not use the warmer cavity for storage.
 - Do not remove exterior panels from the warmer or attempt repairs. The warmer has no user-serviceable internal components. Only perform routine cleaning and maintenance procedures specifically described in this manual. Inspection and servicing of internal components must only be performed by qualified service personnel.
 - Only a qualified Pedigo service representative may make modifications to the warmer. Modifications to the warmer could be hazardous to users and patients.
-

Operator training

All personnel using the warmer must have proper operator training. Before using the warmer:

- Read and understand the operating instructions contained in all the documentation delivered with the warmer.
 - Know the location and proper use of all controls.
 - Keep this manual and all supplied instructions, diagrams, schematics, parts lists, notices, and labels with the warmer if the warmer is sold or moved to another location.
-

Operator qualifications

Only trained personnel with the following operator qualifications are permitted to use the warmer:

- Have received proper instruction on how to use the warmer.
- Are familiar with the purpose, limitations, and associated hazards of the warmer.

The warmer must not be used by:

- People impaired by drugs or alcohol.
-

Condition of warmer

Only use the warmer when:

- All controls operate correctly.
 - The warmer is installed correctly.
 - The warmer is clean.
 - The warmer labels are legible.
-

Servicing the warmer


- Obey precautions in the manual, on tags, and on labels attached to or shipped with the warmer.
 - Only trained personnel are permitted to service or repair the warmer. Repairs that are not performed by a trained technician, or the use of non-factory parts, will void the warranty and relieve all liability.
 - Any troubleshooting guides and components views included with this manual are for reference only and are intended for use by qualified and trained service technicians.
 - To prevent serious injury, death or property damage, have the warmer inspected and serviced at least every twelve (12) months by a trained technician.
 - Contact Pedigo for the authorized service partner in your area.
-

Incident notice

- Any serious incident that has occurred in relation to the warmer should be reported to the manufacturer and the competent authority of the Member State in which the user and/or patient is established.

This page intentionally left blank.

Label Locations

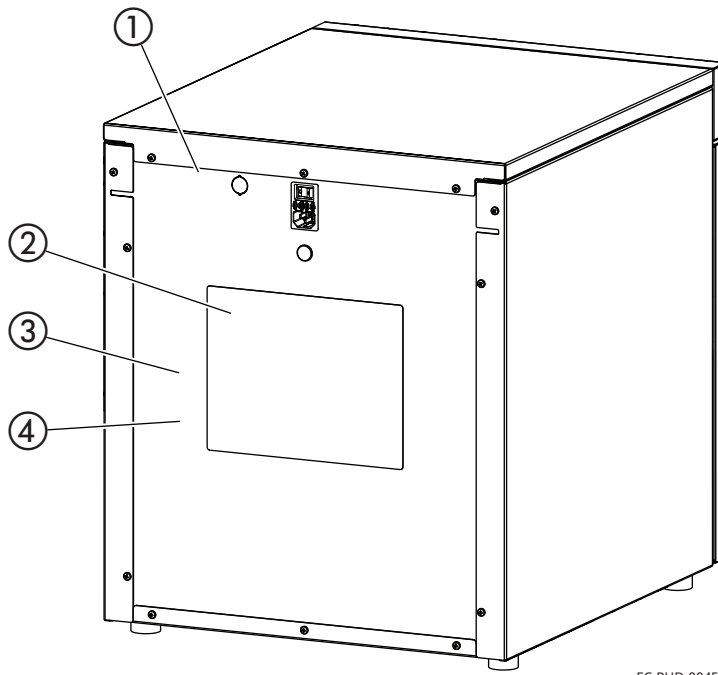

EC-PHD-004557

LABELS

1		Security seal
2	WARNING!  FOR PROTECTION AGAINST FIRE AND ELECTRICAL SHOCK USE ONLY UL LISTED 10A, 250V FAST ACTING FUSES, 5MM X 20MM (F1, F2). ACCESS SHOULD BE MADE BY QUALIFIED SERVICE TECHNICIANS ONLY. HOSPITAL GRADE CORD MUST BE USED. REFER TO OPERATION AND CARE MANUAL OR CONTACT MANUFACTURER FOR ACCEPTABLE CORDS. EQUIPMENT MUST BE CONNECTED TO AN EQUIVALENT RECEPTACLE MARKED "HOSPITAL GRADE". AVERTISSEMENT!  POUR PROTÉGER CONTRE LES INCENDIES ET LES DÉCHARGES ÉLECTRIQUES, UTILISER EXCLUSIVEMENT DES FUSIBLES INSTANTANÉS HOMOLOGUÉS UL DE 10 A, 250 V, 5 MM X 20 MM (F1, F2). L'ACCÈS DOIT ÊTRE RÉSERVÉ EXCLUSIVEMENT À DES TECHNICIENS D'ENTRETIEN QUALIFIÉS. UN CORDON DE QUALITÉ HÔPITAL DOIT ÊTRE UTILISÉ. VOIR LES CORDONS ADMISSEURS DANS LE MANUEL D'UTILISATION ET D'ENTRETIEN OU S'ADRESSER AU FABRICANT. LE MATÉRIEL DOIT ÊTRE RACCORDÉ À UNE PRISE ÉQUIVALENTE MARQUÉE COMME ÉTANT DE « QUALITÉ HÔPITAL ». ¡ADVERTENCIA!  PARA PROTECCIÓN CONTRA INCENDIOS Y DESCARGAS ELÉCTRICAS, USE SOLO FUSIBLES DE ACCIÓN RÁPIDA CON CLASIFICACIÓN UL DE 250 V, 10 A, 5 MM X 20 MM (F1, F2). SÓLO SE DEBE PERMITIR EL ACCESO A TÉCNICOS DE SERVICIO CALIFICADOS. SE DEBE USAR UN CABLE DE CALIDAD PARA USO HOSPITALARIO. CONSULTE EL MANUAL DE OPERACIÓN Y CUIDADO, O BIEN, COMUNIQUESE CON EL FABRICANTE PARA CONOCER LOS CABLES ACEPTABLES. SE DEBE CONECTAR EL EQUIPO A UN TOMACORRIENTE EQUIVALENTE CON LA MARCA "CALIDAD PARA USO HOSPITALARIO". ACHTUNG!  ZUM SCHUTZ VOR BRAND UND STROMSCHLAG NUR EINE SUPERFLINKE SCHMELZSICHERUNG UL LISTED 10A, 250V, 5MM X 20MM (F1, F2) VERWENDEN. DARF DER ZUGANG NUR VON QUALIFIZIERTEN WARTUNGSTECHNIKERN HERGESTELLT WERDEN. ES MUSS EIN FÜR KRANKENHÄUSER GEEIGNETES NETZKABEL VERWENDET WERDEN. FÜR GEEIGNETE NETZKABEL IM BETRIEBS- UND PFLEGEHANDBUCH NACHSCHLÄGEN. GERÄTE MÜSSEN AN EINE STECKDOSE MIT DER BEZEICHNUNG „FÜR KRANKENHÄUSER GEEIGNET“ ANGESCHLOSSEN WERDEN. AVVERTENZA!  PER PROTEZIONE ANTINCENDIO E CONTROLLO LE SCOSSE ELETTRICHE, USA SOLO FUSIBILI AD AZIONE RAPIDA UL LIST 10° DA 250 V, 5 MM X 20 MM (F1, F2). L'ACCESSO DOVEREBBE ESSERE LIMITATO SOLO AI TECNICI DI MANUTENZIONE QUALIFICATI. I DEVE USARE CAVO DI QUALITÀ PER OSPEDALI. VEDERE IL MANUALE OPERATIVO E DI MANUTENZIONE O CONTATTARE IL PRODUTTORE PER SAPERE QUALI SONO I CAVI ACCETTABILI. L'APPARECCHIATURA DEVE ESSERE COLLEGATA AD UN CONTENITORE EQUIVALENTE CONTRASSEGNA TO CON "GRADO MEDICALE". 警告  为了防止火灾与触电，只能使用以认证的 10A、250V 的快速熔断丝，5MM X 20MM (F1, F2)。必须由有资质的维修技术人员进行相关操作。必须采用医疗级电源线。请参考《使用与维修手册》或联系生产商，获取合格电源线。必须将设备连接至标有“医疗级”的等效插座。 LA-29452	Electrical warning label
3	Distributed by:  PEDIGO®  Pedigo Products, Inc. 4000 SE Columbia Way Vancouver, WA 98661 800-822-3501 For Service or Emergencies Call 262-509-6517 ★★★★★ — Made in ★★★★★ — USA ★★★★★ — www.pedigo-usa.com  LA-29565	Pedigo contact information
4	QUALITY INSPECTED _____ INSPECTORS INITIALS _____ DATE LA-36656	Quality inspected

Specification Information

Model	P-2012 P-2210	P-2022 P-2220	P-2230	P-2032 P-2240
Intended use	Warming blankets			
Weight	Net: 67 lb (30 kg) Ship: 116 lb (53 kg)	Net: 91 lb (41 kg) Ship: 144 lb (65 kg)	Net: 86 lb (39 kg) Ship: 144 (65 kg)	Net: 134 lb (61 kg) Ship: 196 lb (89 kg)
Storage cavity capacity	4–6 blankets (2.5 ft ³)	10–12 blankets (3.5 ft ³)	14–18 blankets (4.0 ft ³)	20–25 blankets (7.5 ft ³)
Temperature range	90°F to 160°F (32°C to 71°C)			
Clearance requirements	3" (76mm) from rear 1" (25mm) from top 1" (25mm) from sides 3/4" (19mm) from bottom			

Transportation and storage conditions

- Ambient temperature range of -40°F to +159°F (-40°C to +70°C).
- Relative humidity range of 10% to 95%, non-condensing.
- Atmospheric pressure range of 7.25 psi to 15.37 psi (50kPa to 106kPa).

Operating conditions

- The warmer must acclimate to the room temperature it will be placed in—24 hours is recommended.
- The recommended environmental temperature range is 60°F to 90°F (15°C to 32°C).
- The recommended relative humidity is above 20%, non-condensing.


Standards for electrical equipment

- Medical equipment listed by Underwriters Laboratories with respect to electrical shock, fire, and mechanical hazards only, in accordance with UL 61010-1 and CAN/CSA C22.2 No. 61010-1.
- Grounding reliability can only be achieved when the appliance is connected to an equivalent receptacle marked "Hospital Grade."
- IP-XO ordinary




Dimension Drawings

P-2012 / P-2210 shown with Standard Feet

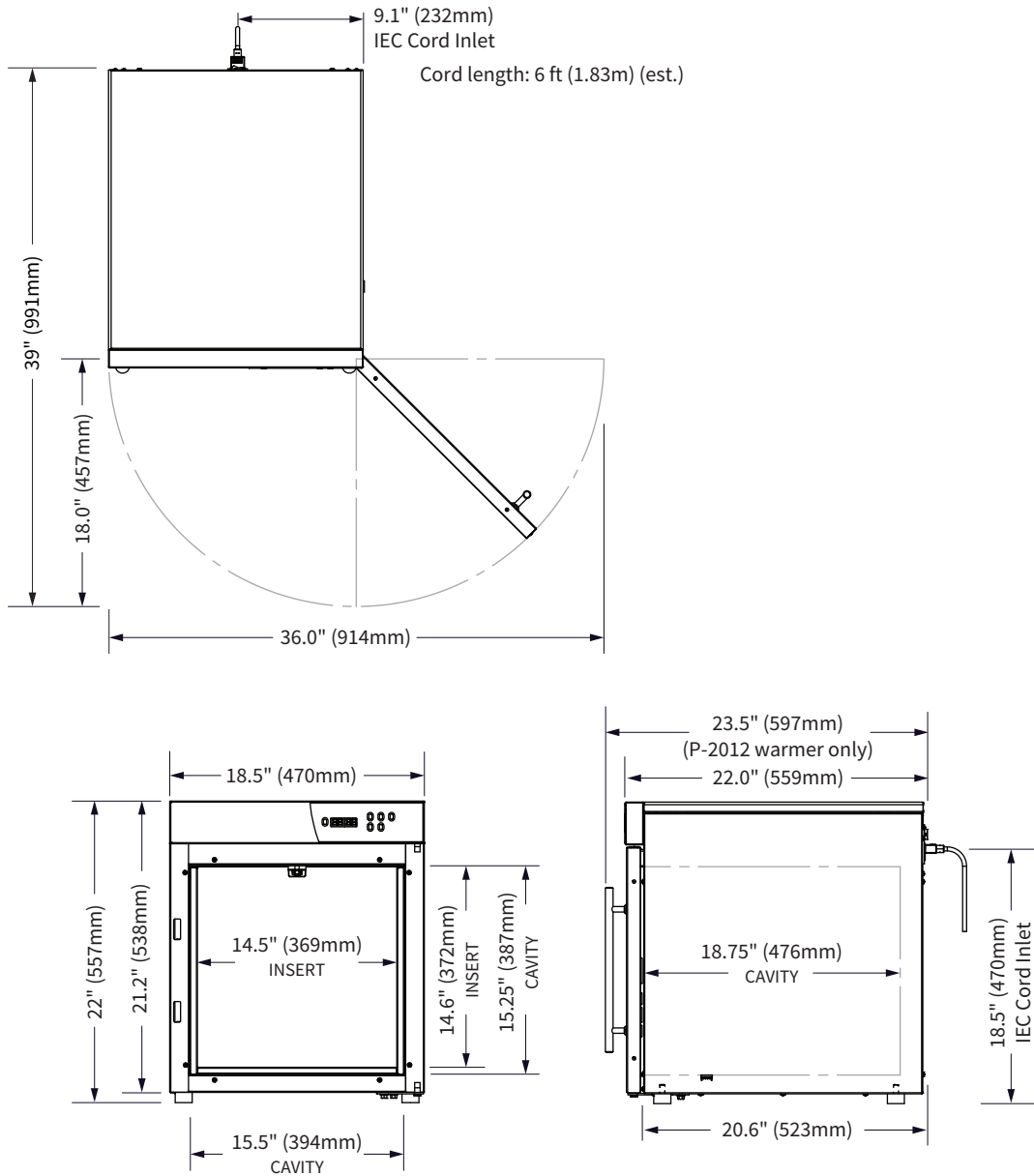


EC-DIM-004322

P-2012 / P-2210 shown with Optional Plate and Casters

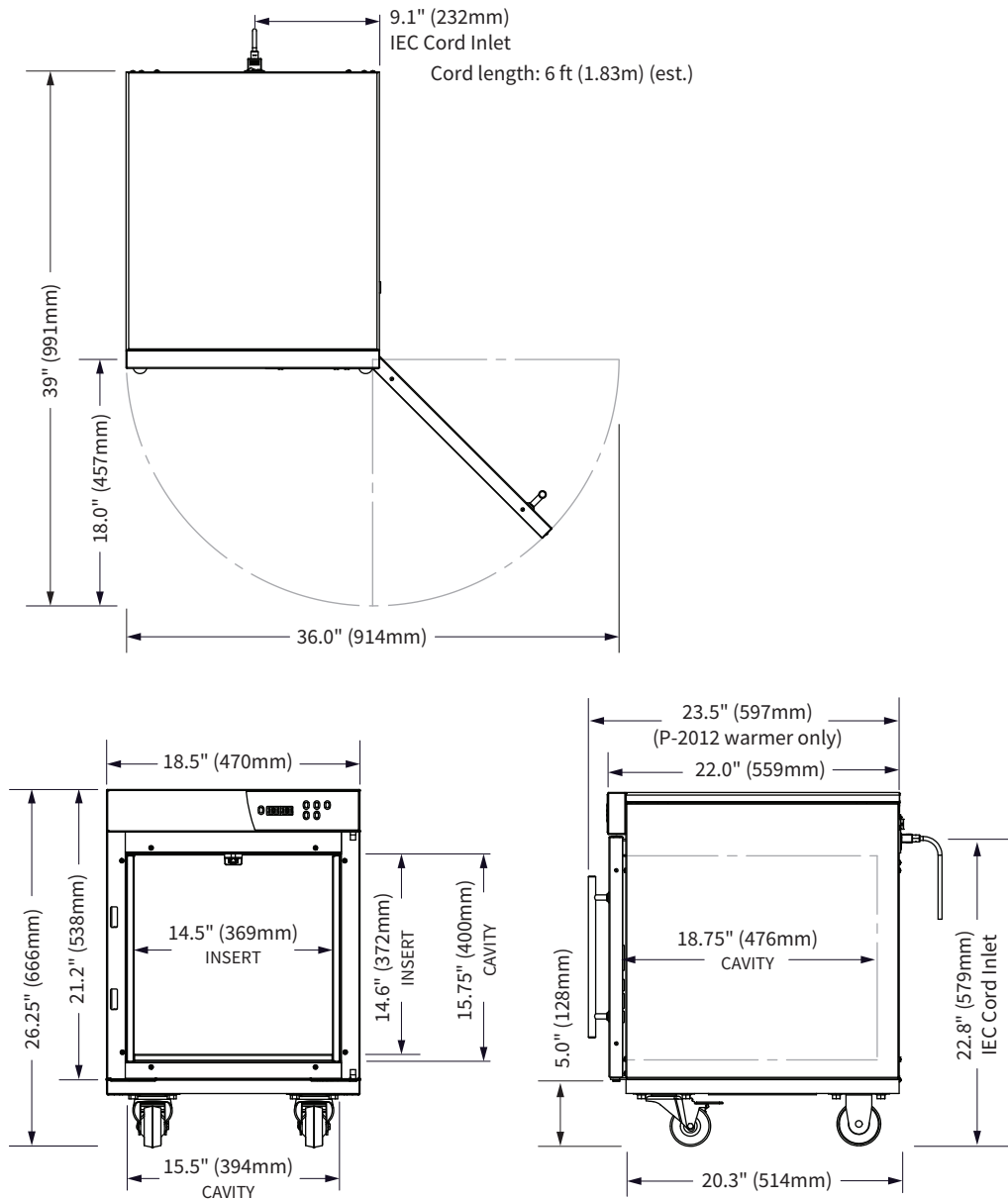




EC-DIM-004325

P-2012 / P-2210 shown with Optional Bumpers and Casters

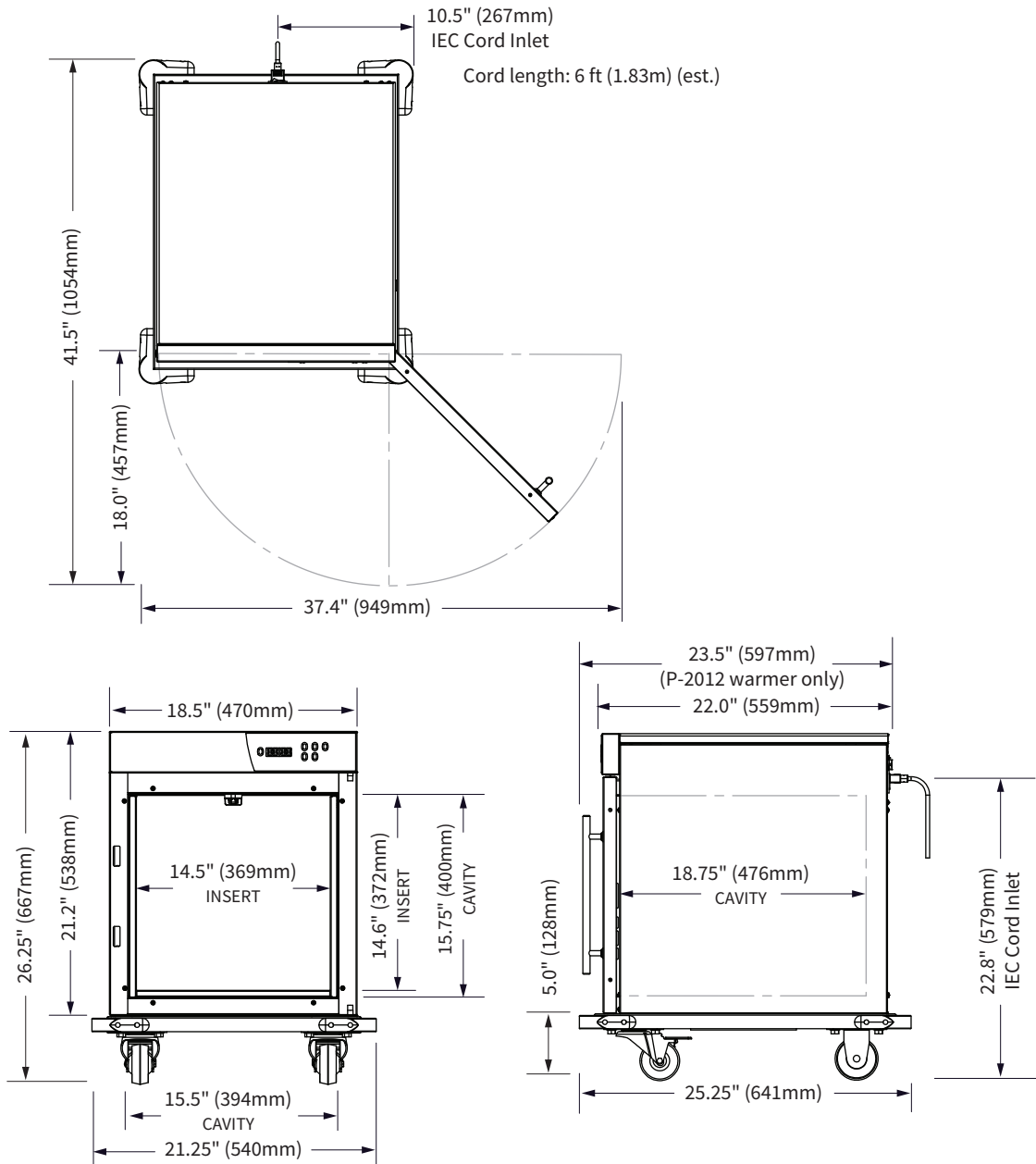



EC-DIM-004328

P-2022 / P-2220 shown with Standard Feet

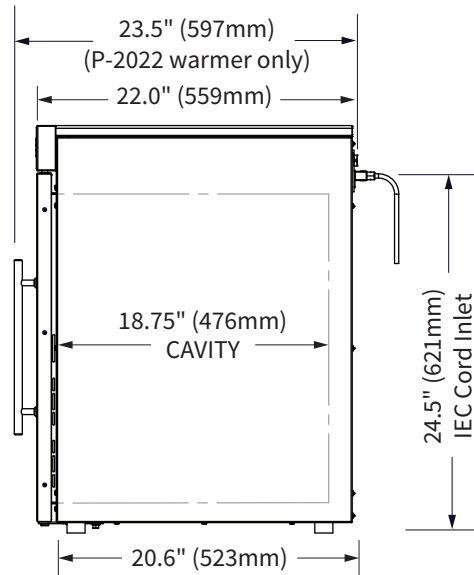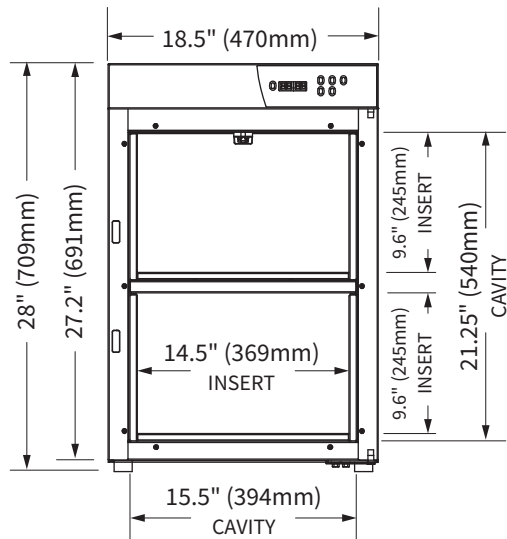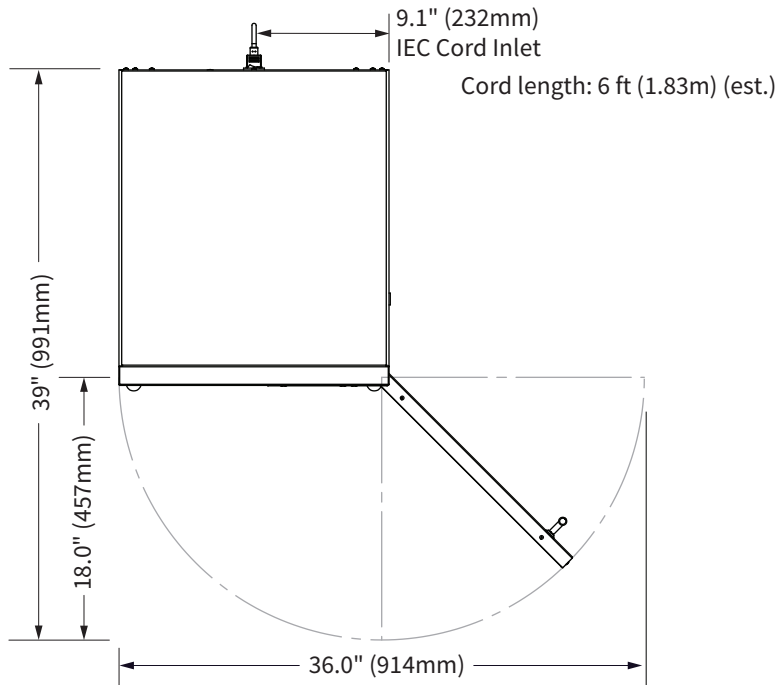


EC-DIM-004331

P-2022 / P-2220 shown with Optional Plate and Casters

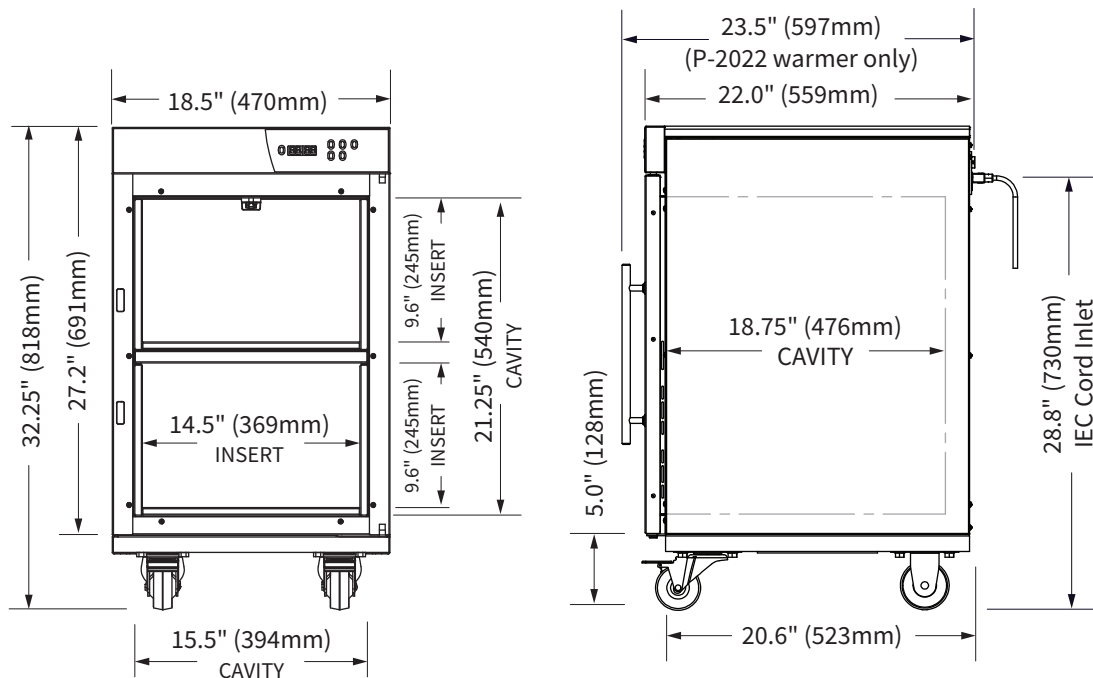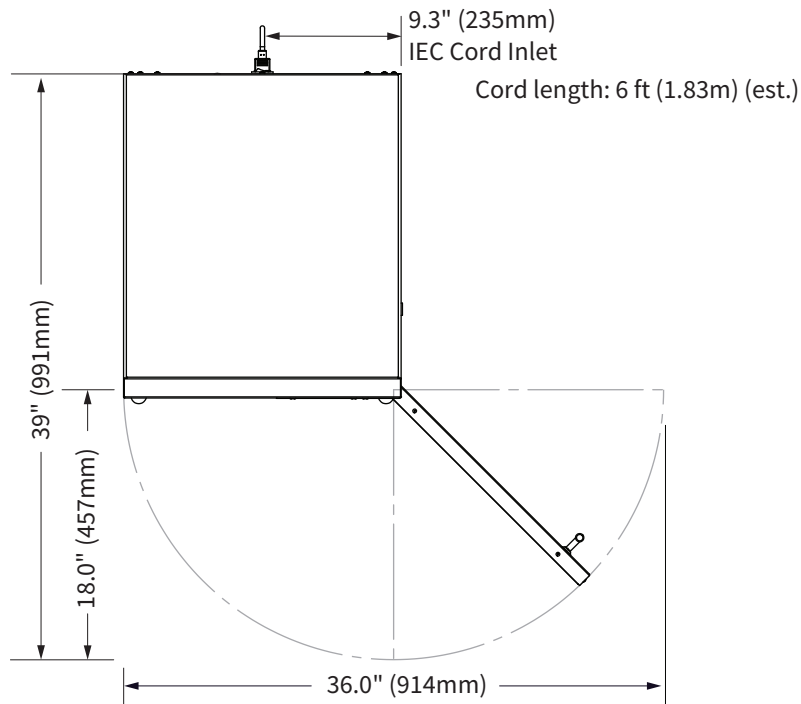


EC-DIM-004334




P-2022 / P-2220 shown with Optional Bumpers and Casters

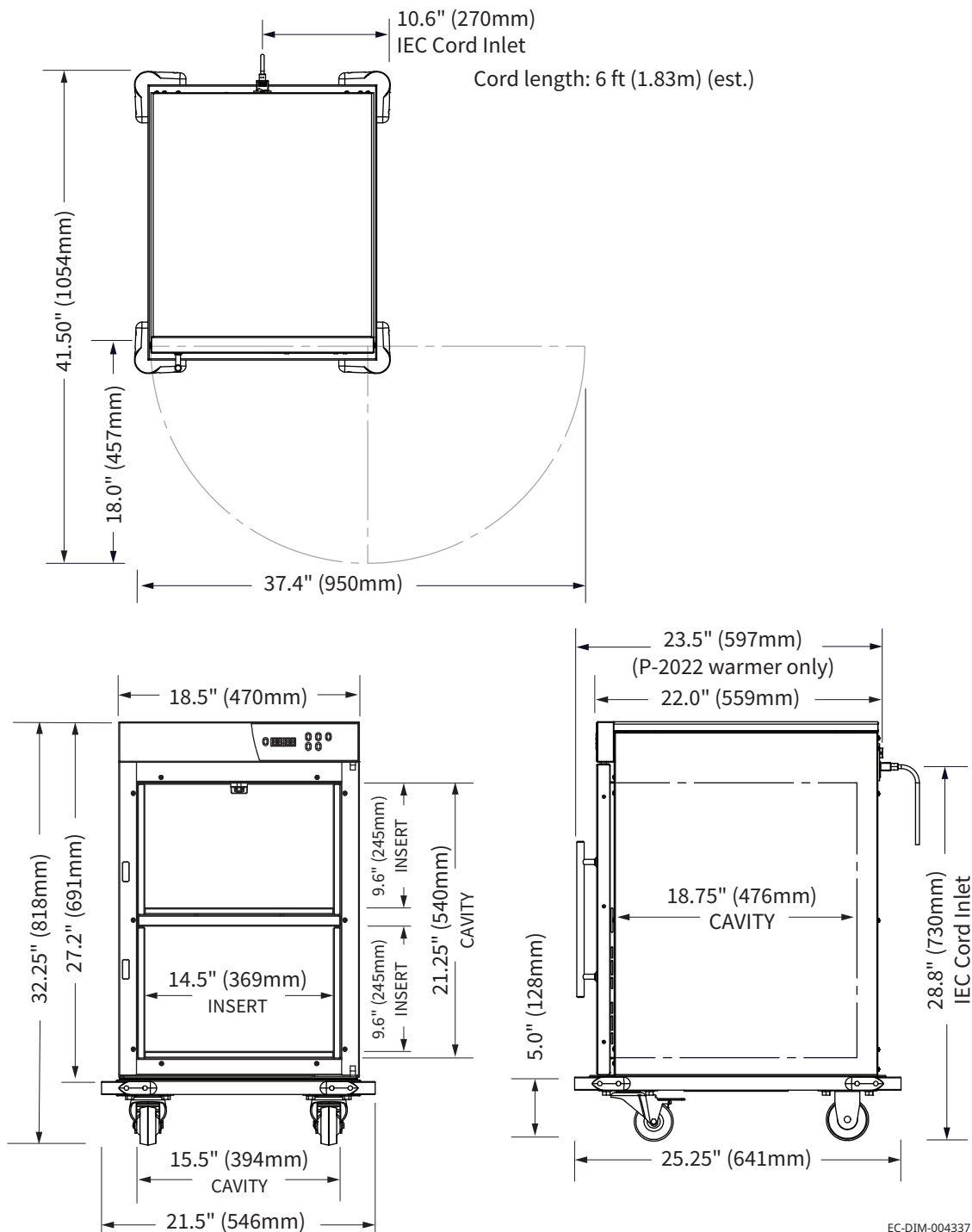

P-2230 shown with Standard Feet

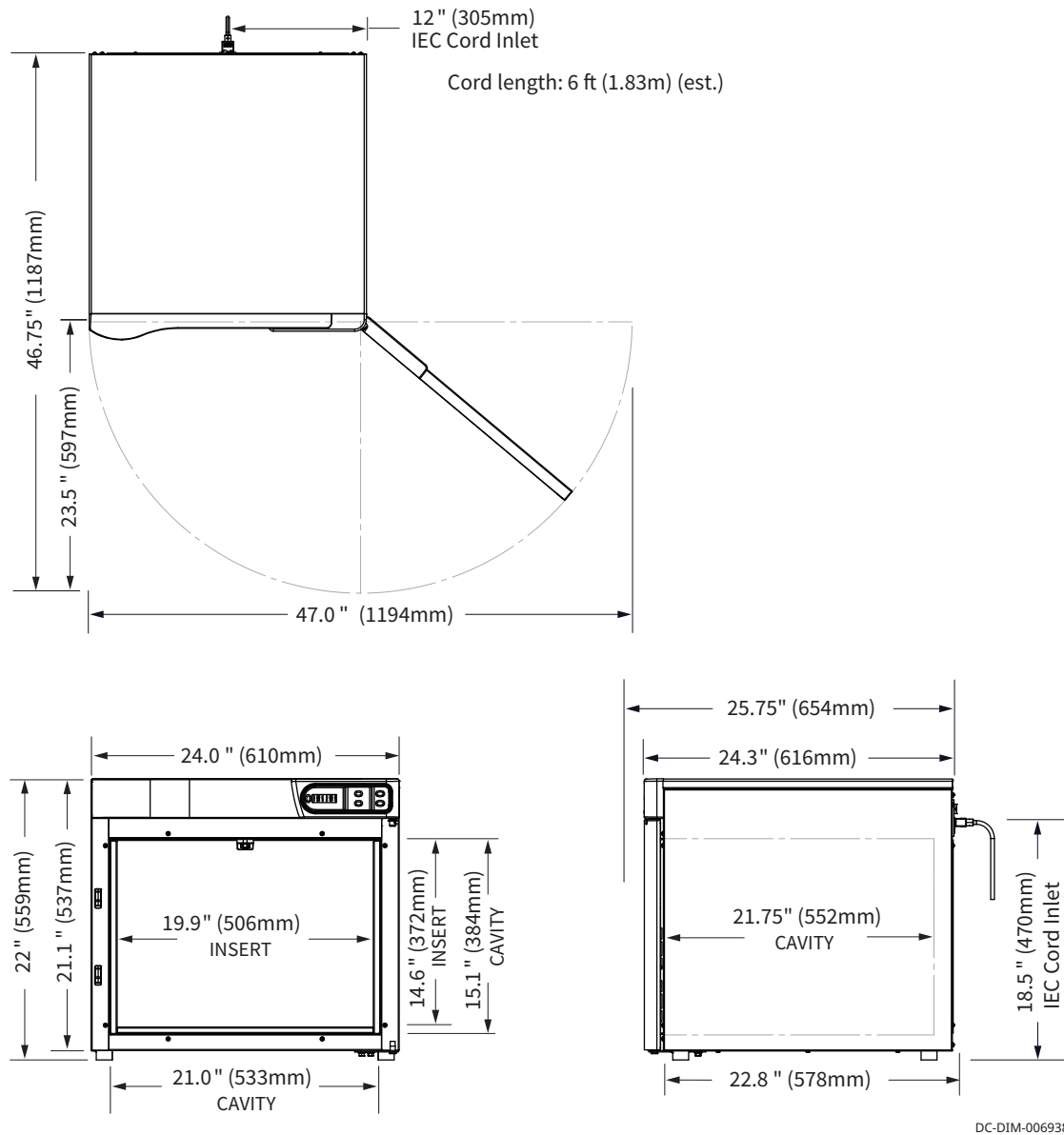


P-2230 shown with Optional Plate and Casters

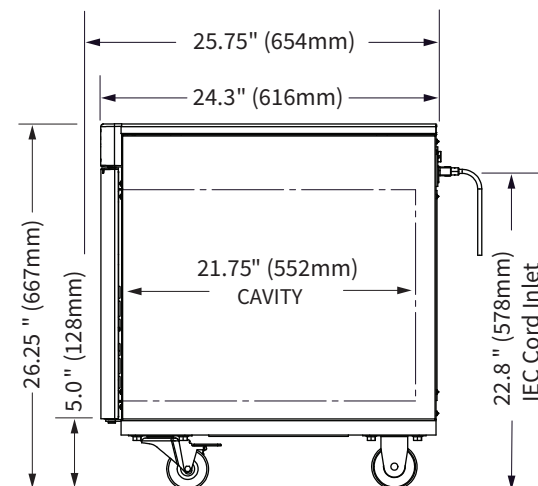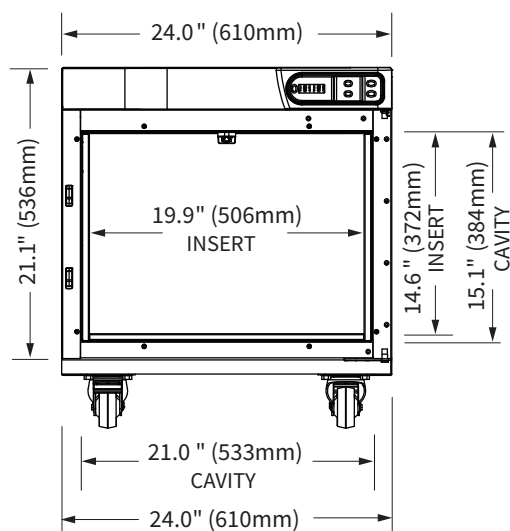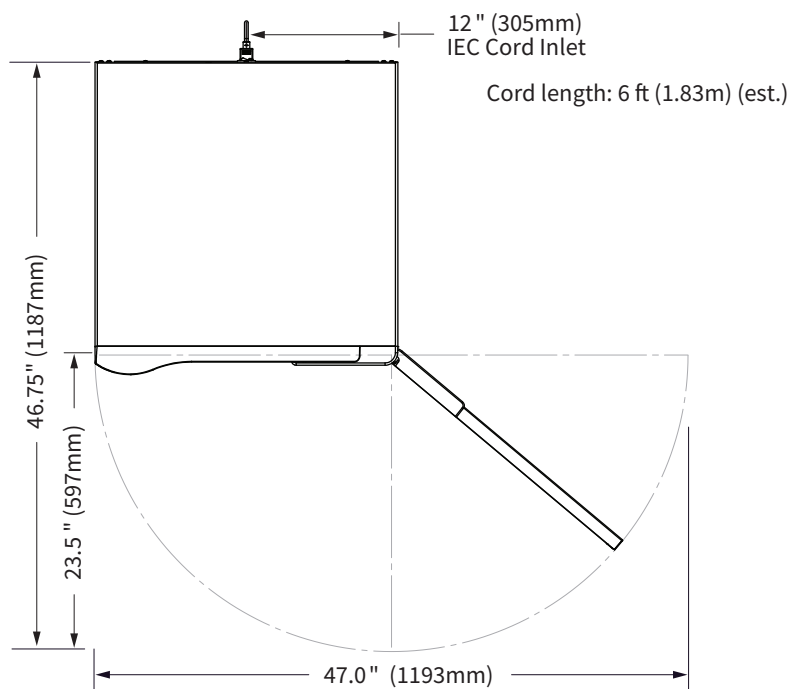


DC-DIM-006941

P-2230 shown with Optional Bumpers and Casters

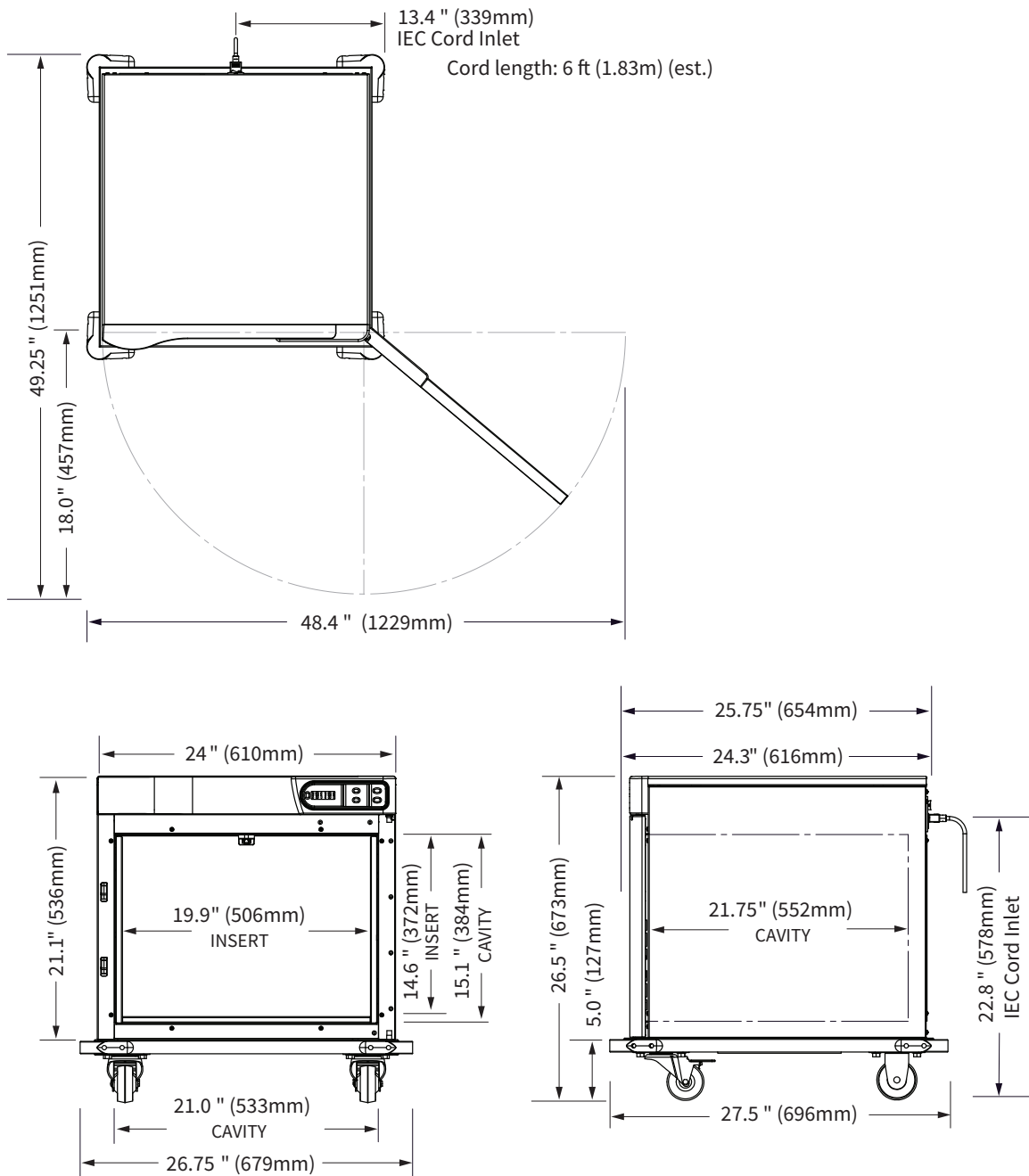


DC-DIM-006944

P-2032 / P-2240 shown with Standard Feet

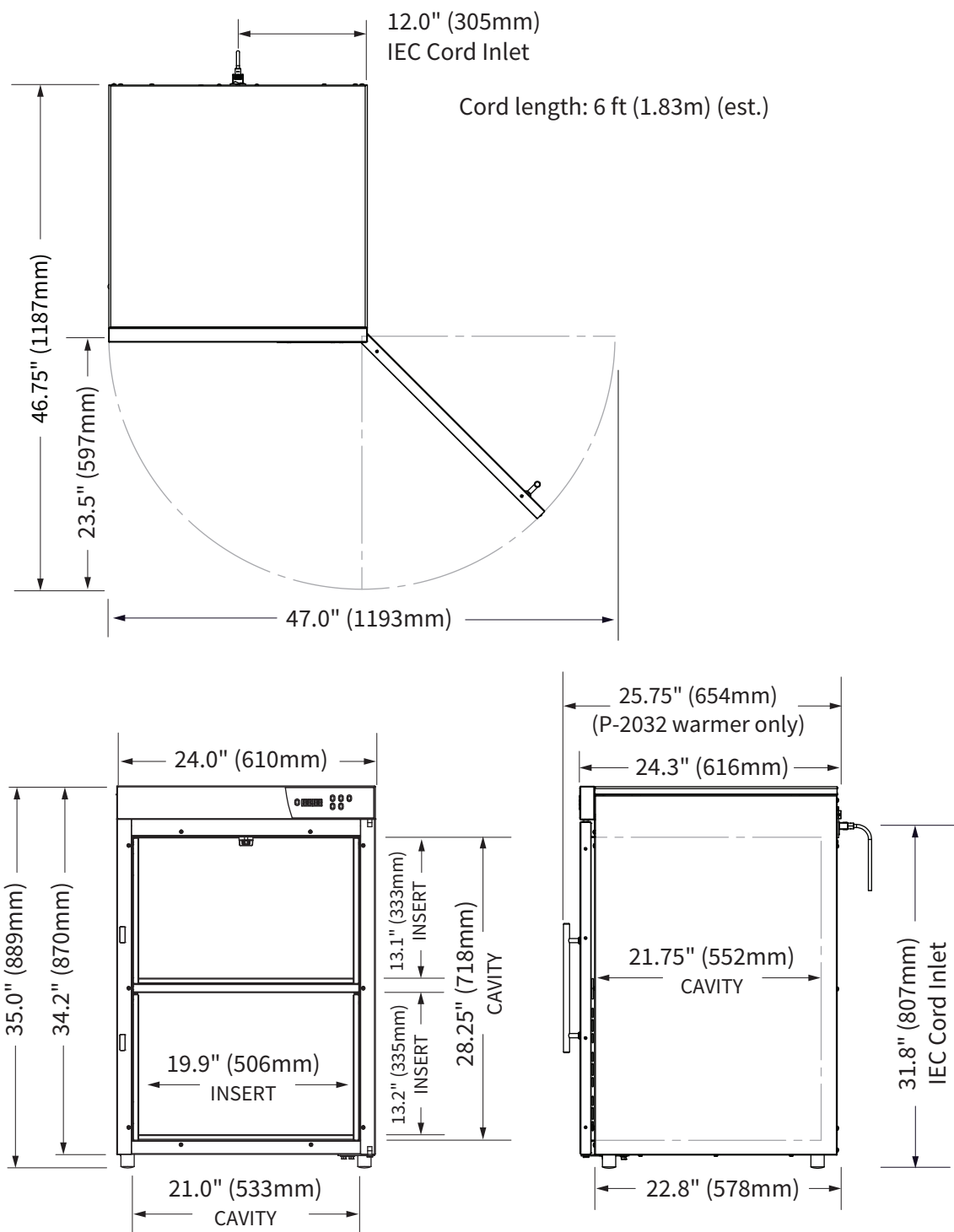


FC-DIM-004240


P-2032 / P-2240 shown with Optional Bumpers and Casters

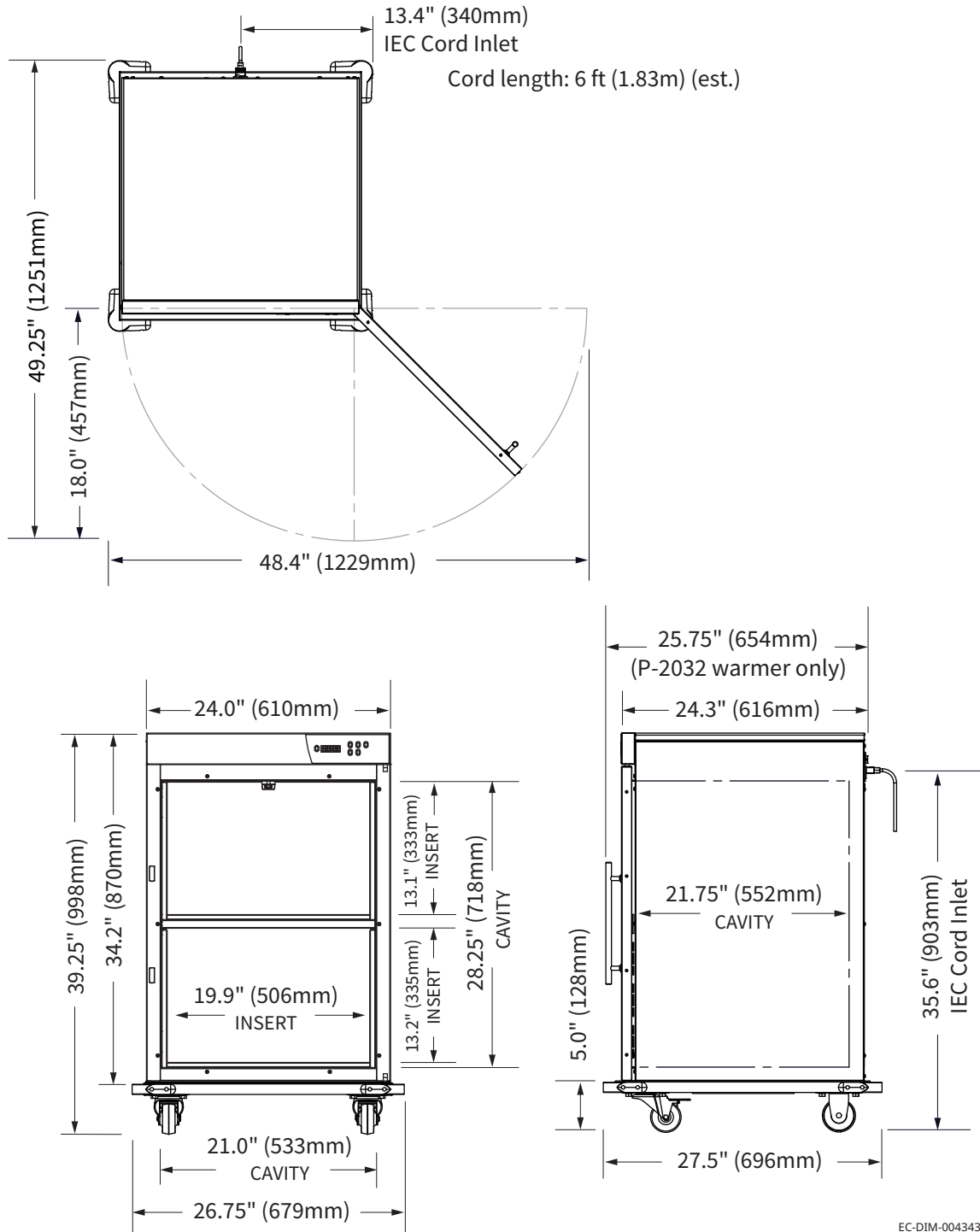


P-2032 / P-2240 shown with Optional Casters

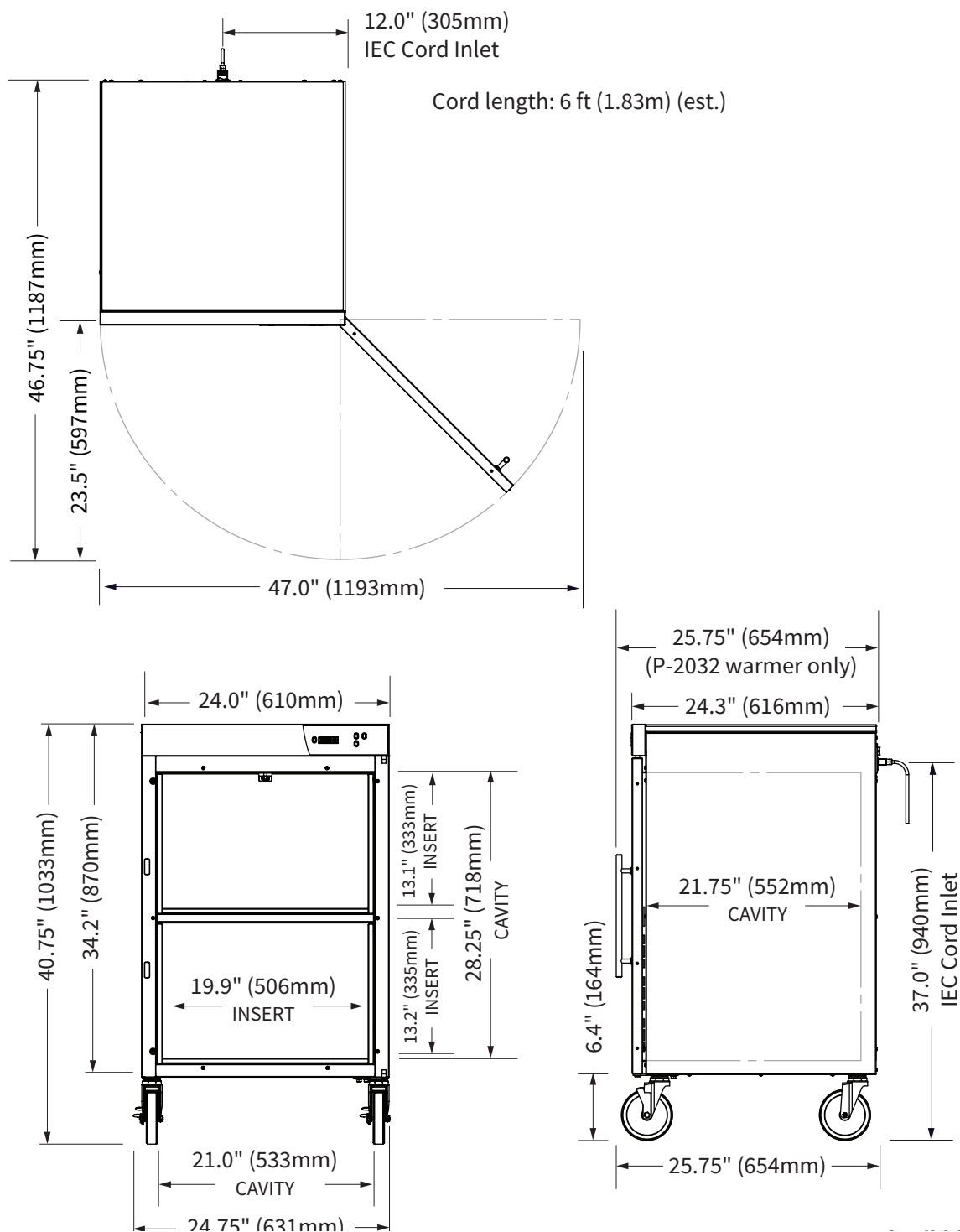

P-2032/PT shown with standard Feet

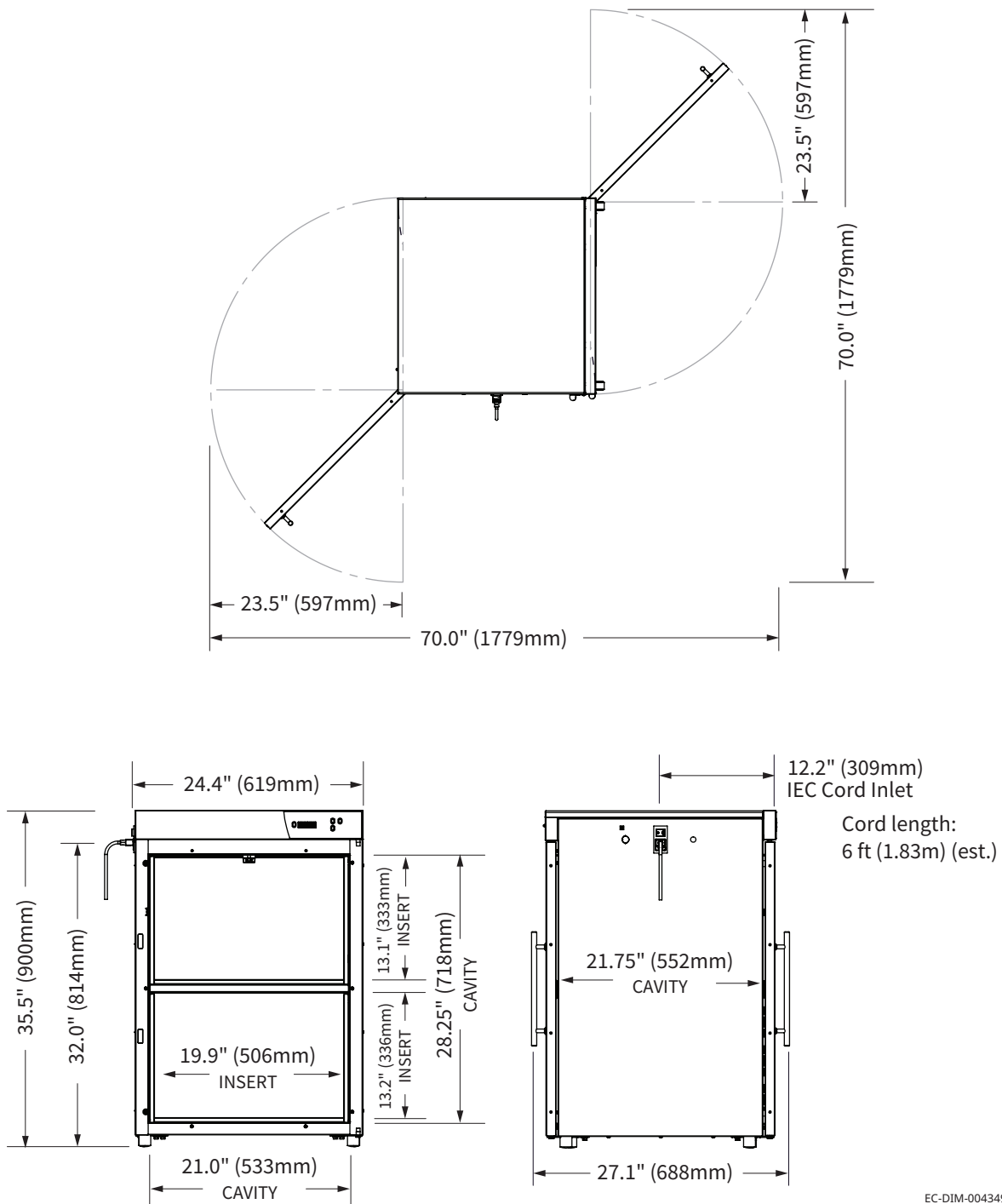

P-2032/PT shown with Optional Bumpers and Casters

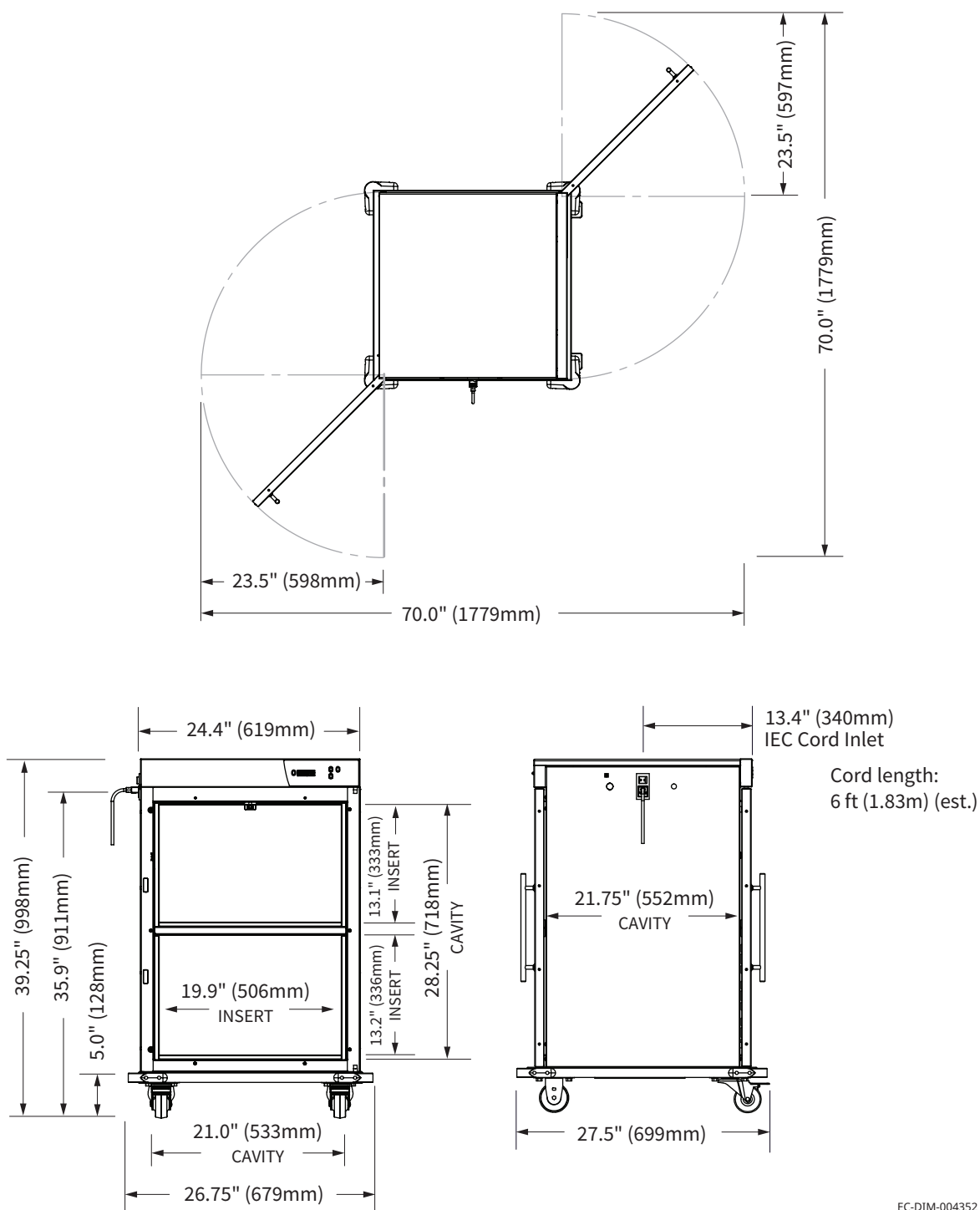



How to Unpack the Warmer



WARNING: Crushing hazard.

An unstable warmer can lead to a crushing hazard. Observe your facility's best practices for moving large equipment.

Before you begin

Make sure you have:

- Reviewed the warmer specifications.
- An appropriate lifting device and enough personnel to safely move and position the weight of the warmer.
- Cutting tools to remove the packaging.

Unpack the warmer

To unpack the warmer, do the following.

Step	Action
1.	Remove the box or crate. Save all packing materials for inspection by the carrier.
	 NOTE: Examine the warmer for damage. If the warmer has been damaged, do not use the warmer until it has been inspected by an authorized service provider. Contact your carrier or customer service.
2.	Cut and remove the retaining straps and plastic wrap.
3.	Remove the warmer from the pallet using an appropriate lifting device.
4.	Remove the paperwork from the cavity.

Result

The warmer is now unpacked.

How to Install the Warmer

Before you begin

Make sure you have:

An appropriate lifting device and enough personnel to safely move and position the weight of the warmer.

Requirements

- The warmer must be installed on a level surface.
- The warmer must not be installed in any area where it may be affected by steam, dripping water, high temperature, or any other severely adverse conditions.

Voltages

Model	V	Ph	Hz	kW	Plug Configuration
P-2012 P-2210	120	1	60	0.6	NEMA 5-15P
P-2022 P-2220	120	1	60	0.8	NEMA 5-15P
P-2230	120	1	60	0.6	NEMA 5-15P
P-2032 P-2240	120	1	60	0.8	NEMA 5-15P

Position the warmer

To position the warmer, do the following.

Step	Action
1.	Make sure that: ■ The location where the warmer is being installed is rated to support the weight of the warmer; ■ The warmer is within five feet of the appropriate electrical outlet; ■ You follow the warmer clearance guidelines. Refer to topic <i>Specification Information</i> for clearance requirements.
2.	Move the warmer to the installation location and onto the final resting surface.
3.	Lock the casters, if equipped.

The warmer is now correctly positioned.


Continued on next page

Continued from previous page

Connect power

To connect electric power to the warmer, do the following.

4. **Connect** the plug to the electrical outlet.

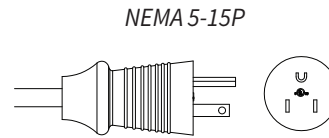

EC-TS-005221

Result

The warmer is now installed and ready to be used.

This page intentionally left blank.

Preparing the Warmer for First Use

Before you begin



CAUTION: Burn hazard.
Allow the warmer to cool before cleaning.

NOTICE

Do not use:

- abrasive cleaning compounds.
- chloride based cleaners.
- commercial or household cleaners containing ammonia.

Procedure

To prepare the warmer for first use, do the following.

Step	Action
1.	Make sure that the warmer is turned off and cool.
2.	Wipe the outside of the warmer: ■ with a stainless steel cleaner, if stainless steel panels. ■ with an approved cleaning agent, if painted panels.
3.	Clean the interior of the warmer with a damp cloth or approved cleaning agent.
4.	Dry the interior and exterior of the warmer with a clean, lint-free cloth. Leave the door open until the interior of the warmer completely dries.
5.	Clean each side of the window pane with glass cleaner or distilled vinegar.

Result

The warmer is now ready for use.

How to Turn On and Turn Off the Warmer

Before you begin


The warmer must be connected to electric power.

Turning on the warmer

To turn on the warmer, do the following.

Step	Action
1.	Set the power switch ① to the ON (I) position.

1. **Set** the power switch ① to the ON (I) position.

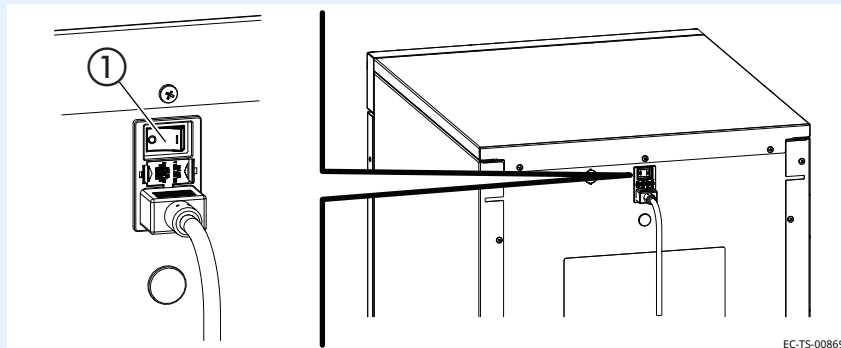

2. **Press** the standby button . The screen turns on.

NOTE: A temporary odor may be noticeable upon initial start-up of the warmer. Contact the manufacturer if the odor persists after a day or more of continuous use.

The warmer is now on.

Turning off the warmer

To turn off the warmer, do the following.

3. **Press and hold** the standby button  until the screen turns off, then release the button.

The warmer is now off.

How to Operate the Blanket Warmer

Before you begin

The warmer must be connected to electric power and turned on.



WARNING: Personal injury hazard.

Do not operate the warmer in the presence of a flammable anesthetic mixture with air, oxygen, or nitrous oxide; in oxygen-enriched environments; or in any other potentially explosive environment.



WARNING: Personal injury hazard.

Make sure the blanket support assembly and shelves are installed to prevent the blankets from being scorched or discolored.

Overloading or covering the sensor can potentially cause an error code and sensor may fail prematurely.

NOTICE

Do not overload the cavity.

Allow 1" (25mm) between the blankets and the interior walls and 3" (76mm) between the top of the blankets and the sensor.

Procedure

To operate the warmer, do the following.

Step	Action
1.	Set the temperature using the arrow buttons   .  NOTE: The temperature set-point range is 90°F – 160°F (32°C – 71°C).
2.	Open the door and load the dry, cotton blankets into the warmer. Close the door.  NOTICE Do not warm items containing plastic, rubber, metal snaps, studs, hooks, etc.
3.	Press the temperature recall button  to view the measured cavity temperature. The measured cavity temperature will display for five seconds. Then, the set-point temperature will display.
4.	Rotate the blankets daily from the bottom to the top of the load to ensure equal heating and to prevent discoloration.

During the warming process

Result

The blankets are now warming.

How to Change the Temperature Scale

Before you begin

The warmer must be connected to electric power and the screen turned off.

Procedure

To change the temperature scale from °F to °C and vice versa, do the following.

Step	Action
1.	Press and hold the standby button  until the screen turns off, then release the button.
2.	Press and hold the temperature recall button  until the temperature digits disappears and only the F or C display, then release the button.
3.	Press the up and down arrow buttons   to toggle between the temperature scales.
4.	Press the standby button  to turn the screen on.

Result

The temperature scale has now been changed.

How to Change the Sound Settings

Before you begin

The warmer must be connected to electric power.

Procedure

To change the sound settings, do the following.

Step	Action
1.	Press and while holding the temperature recall button  , press the down arrow button  . Once the sound volume setting displays, release both buttons.
2.	Press the up and down arrow buttons   to adjust the volume. The volume range is 0 (mute) to 12 (maximum).
3.	Press the standby button  to turn the screen on.

Result

The sound setting has now been changed.

How to Lock and Unlock the Controller

Before you begin

The warmer must be turned on (screen is on).

Background

The controller can be locked to prevent changes being made to the temperature set-point.

Locking the controller

To lock the controller, do the following.

- | Step | Action |
|------|--|
| 1. | Press and while holding the standby button  , press the up arrow button  .
The lock indicator  will illuminate. |



NOTE: Make sure the set-point temperature did not change during the locking process.

Unlocking the controller

To unlock the controller, do the following.

- | | |
|----|---|
| 2. | Press and while holding the standby button  , press the down arrow button  .
The lock indicator  goes off. |
|----|---|



NOTE: Make sure the set-point temperature did not change during the unlocking process.

Result

The controller has now been locked or unlocked.

How to Adjust the Interior Light (if equipped)

Before you begin

The warmer must be turned on (screen is on).

Procedure

To adjust the interior light, do the following.

Step	Action
1.	Press the interior light button . Press the interior light button again to adjust between the light settings.
	 NOTE: The interior light has three settings: high, low, and off.

Result

The interior light has now been adjusted.

Maintenance Schedule



NOTE: Do not remove exterior panels from the warmer or attempt repairs. The warmer has no user-serviceable internal components. Only perform routine cleaning and maintenance procedures specifically described in this manual.

Daily

For daily maintenance, do the following.

Check:

- the air vents in the airflow insert panels are not obstructed (if applicable).
- all fan guards are clear and not obstructed (if applicable).
- the number of bottles/bags, or blankets as applicable do not exceed the maximum capacity per shelf or basket.

Clean:

- any spills with a clean, lint free cloth. See topic *How to Clean the Warmer* for the list of approved cleaners.

Monthly

For monthly maintenance, do the following.

Check:

- the door gasket for tears and holes. Make sure that it is firmly attached to the door. Check the seal when the door is closed.
- the guards around the air temperature sensor are in place and fully secure to the warmer.
- the hardware securing the warmer(s) to the wall, if applicable.

Clean:

- the outside of the warmer with stainless steel cleaner. See topic *How to Clean the Warmer* for the list of approved cleaners.
- vacuum (if applicable):
 - fan openings
 - fan sail switch
 - vent openings

Continued on next page

Continued from previous page

Yearly

For yearly maintenance, do the following.

Check:

- the set-point temperature compared to the actual temperature displayed.
 - Check the cavity air temperature with a thermocouple placed 1" (25mm) from the cavity sensor. Do not allow the thermocouple to touch any surface. Monitor the temperature for approximately one hour in an empty cavity.
- the condition of the plug and cord and replace if damaged.
- the controller screen for excessive wear. Make sure the controller screen displays and operates properly.

Clean:

- the shelves and interior of the warmer. See topic *How to Clean the Warmer* for the list of approved cleaners.

How to Clean the Warmer

Before you begin



WARNING: Electric shock hazard.
Disconnect the warmer from electric power before cleaning.



CAUTION: Burn hazard.
Allow the warmer to cool before cleaning.

NOTICE

Do not use:

- abrasive cleaning compounds.
- chloride based cleaners.
- commercial or household cleaners containing ammonia.
- cleaners containing quaternary salts.

Monthly cleaning procedure

To clean the warmer monthly, do the following.


Step	Action
1.	Make sure that the warmer is disconnected from electric power and cool.
2.	Wipe the outside of the warmer: ■ with a stainless steel cleaner, if stainless steel panels. ■ with an all purpose cleaner, if painted panels.
3.	Vacuum lint from fan openings, fan sail switch, and vent openings (if applicable).
4.	Clean the interior of the warmer with a damp cloth.
5.	Dry the interior and exterior of the warmer with a clean, lint-free cloth. Leave the door open until the interior of the warmer completely dries.
6.	Clean each side of the window pane with glass cleaner or distilled vinegar.

Continued on next page

Yearly cleaning procedure

Continued from previous page

To clean the warmer yearly, do the following.

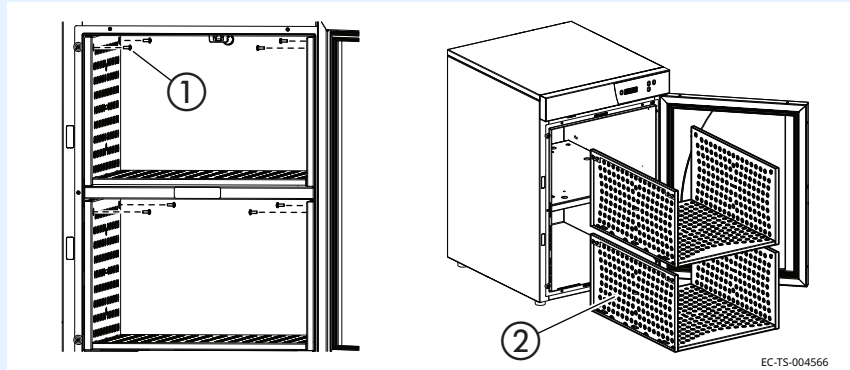Step	Action
1.	Make sure that the warmer is disconnected from electric power and cool.
2.	Remove the screws ① from the support assembly. Remove the support assembly ② from the interior of the warmer. 
3.	Clean the support assembly and interior with a damp cloth.
4.	Dry the interior of the warmer with a clean, lint-free cloth. Leave the door open until the interior of the warmer completely dries.
5.	Clean each side of the window pane with glass cleaner or distilled vinegar.
6.	Re-install the support assembly.  WARNING: Personal injury hazard. The blankets may overheat if the support assembly is not in place. Make sure to re-install the support assembly before operating the warmer.

Result

The warmer is now clean.

Warmer Disposal / Decommissioning

This product and its accessories must be disposed of according to local laws and regulations. Do not dispose of this product as unsorted municipal waste.

What to do if a Power Interruption Occurs

Background You may need to reset the warmer in the case that a power interruption occurs. When the power is restored, the ON/OFF status indicator LED decimal flashes **68F**.

The controller stores all settings and continues operation using these settings in the event of a power interruption.

Procedure To continue operation of the warmer, do the following.

Step	Action
1.	Press the standby button  . The status indicator LED decimal goes out. If an error code displays, refer to topic <i>Error Codes</i> .
2.	Make sure the blankets are at the correct temperature before using.

Result Resume operation of the warmer.

What to do if the Alarm Indicator Light Flashes

Background

The alarm indicator light  flashes and the controller sounds an alarm when the warmer malfunctions.

Procedure

If the alarm indicator light flashes, do the following.

Step	Action
1.	Press the standby button  to acknowledge and mute the alarm. The alarm indicator light goes off. Refer to the troubleshooting section for the error code.
2.	Make sure the blankets are the correct temperature before using.

Result

Resume operation of the warmer.

Error Codes

Background

This section is provided for the assistance of qualified and trained service technicians only and is not intended for use by untrained or unauthorized service personnel. Failure to observe this precaution may void the warranty.



NOTE: If the warmer is not operating properly, check the following before calling an authorized service agent:

Verify that the power to the warmer is on.

If applicable, ensure the female end of plug is securely seated in the warmer and that the male end of plug is in an appropriate, functioning outlet.

If applicable, examine the fuses. Replace the fuses, refer to topic *How to Replace a Fuse*.

If applicable, examine the high limit manual reset button. If the high limit manual reset button is tripped, reset the warmer, refer to topic *How to Manually Reset the Warmer*.



NOTE: All non-critical codes can be cleared using the standby button. Critical errors (marked with a *) can only be cleared by setting the power switch at the rear of the warmer to the off (O) position and allowing the warmer to cool.

NOTICE

Do not attempt to repair or service the warmer beyond this point. Contact the manufacturer for the nearest authorized service agent. Repairs made by any other service agent without prior authorization by the manufacturer will void the warranty.

Code	Refers to	Action required
Display flashes set point	Cavity temperature higher than set point	Cavity temperature is higher than the set point temperature. Allow cavity to cool to set point temperature.
E-10 ES10 ES20 ES30 ES40 ES50 ES60 ES70	Cavity sensor Sensor 1 Sensor 2 Sensor 3 Sensor 4 Sensor 5 Sensor 6 Sensor 7	Sensor is shorted. Software disengages heating pads. Acknowledge error by pressing the standby button. If error persists, a qualified service technician should test the sensor. - Test the sensor. Detach the sensor from the warmer. Use an Ohm meter to measure the resistance of the sensor. Check the sensor at 25°C (77°F). If the reading is 10 KOhm $\pm$ 1.5 KOhm, replace the display. If the reading is $\pm$ 2 KOhm, replace the sensor. - Check the wires for integrity. Inspect the connections at the control and terminal block to ensure proper and secure connections. If necessary, re-secure the faulty connections. - Contact service if error persists.

Code	Refers to	Action required
E-11 ES11 ES21 ES31 ES41 ES51 ES61 ES71	Cavity sensor Pad sensor 1 Pad sensor 2 Pad sensor 3 Pad sensor 4 Pad sensor 5 Pad sensor 6 Pad sensor 7	Sensor is open. Software disengages heating pads. Acknowledge error by pressing the standby button. If error persists, a qualified service technician should test the sensor. ■ Test the sensor. Disconnect the sensor from the warmer control PCB and use an Ohm meter to measure the resistance of the sensor. Sensor PR-37140 should measure $10\text{ KOhm} \pm 1.5\text{ KOhm}$ at 25°C (77°F). Sensor SN-33541 should measure $100\text{ Ohm} \pm 10\text{ Ohm}$ at 25°C (77°F). Replace sensor if measurement is outside of tolerance. ■ Check the wires for integrity. Check for proper and secure connections at the control and terminal block. If necessary, re-secure the faulty connections. ■ Contact service if error persists.
E-30	Under temperature	The cavity temperature is lower than the set temperature for 90 minutes or longer. ■ Make sure the door is closed. ■ If the cavity is overloaded, redistribute the inventory. Do not exceed the height of the insert. ■ Test the sensor. Disconnect the sensor from the warmer control PCB and use an Ohm meter to measure the resistance of the sensor. Sensor PR-37140 should measure $10\text{ KOhm} \pm 1.5\text{ KOhm}$ at 25°C (77°F). Sensor SN-33541 should measure $100\text{ Ohm} \pm 10\text{ Ohm}$ at 25°C (77°F). Replace sensor if measurement is outside of tolerance.
*E-31	Cavity sensor	■ The sensor reading is above the temperature set-point. Blanket warming mode triggers at 15° over set-point. Fluid warming mode triggers at 5° over set-point. ■ The difference between the room temperature and the fluid set-point temperature must be greater than 11°C (20°F). ■ Contact service if error persists.
P131 P231 P331 P431 P531 P631 P731	Pad sensor 1 Pad sensor 2 Pad sensor 3 Pad sensor 4 Pad sensor 5 Pad sensor 6 Pad sensor 7	Heater pad over-temp error. ■ Software disengages heating pads. ■ Acknowledge error by pressing the standby button. ■ Allow the warmer to cool. ■ Contact service if the error persists.
*E-33	Cavity sensor	■ Sensor reading is above maximum allowable temperature set-point and over temp value. Blanket warmers trigger at 82°C (180°F). Fluid warmers trigger at 71°C (160°F). ■ Contact service if error persists.
*E-50	Analog to Digital Converter Error	■ Remove inventory, discard if necessary, and allow the warmer to cool down. ■ If error persists after cool down and reset, the control assembly should be replaced by a qualified service technician. Contact service.

Code	Refers to	Action required
E-60	Real Time Clock Checksum Error (Blanket warmers only)	Real time clock rechargeable battery backup has discharged. ■ Plug the warmer into the outlet for 30 minutes.
*E-61	Real Time Clock (Blanket warmers only)	Real time clock not responding. Contact service if error persists.
E-62	Real Time Clock (Blanket warmers only)	Timer overlay is present, but no real time clock is detected. Contact service.
*E-70	Pad Count Error	More heater pads detected than set for. Hold the standby button for 12 seconds until display shows "PAd#" (# = number of pads selected [3-7]). Press up or down arrow to adjust to correct number of pads. [Blanket warmers: 1.5ft ³ , 2.5ft ³ = 3 pads, 3.5ft ³ , 4.0ft ³ , 7.5ft ³ = 4 pads. Fluid warmers: 2.5ft ³ & 4.0ft ³ = 3 pads].
*E-71	Personality Error	Contact service.
E-80	EEPROM Error	EEPROM not responding. Contact service if error persists.
*E-81	Calibration not locked	Contact service.
*E-83	EEPROM Error	Contact service for help resetting the control.
E-87	EEPROM Error	Stored offsets corrupted. Offsets reset to 0. ■ Control may need to be re-calibrated. ■ Possible bad EEPROM. ■ Contact service if error persists.
E-90	Button stuck	A button has been held down for >60 seconds. Adjust control. Error will reset when the problem has been resolved.
E-95	Factory Test pin short detected.	Ensure that debris is not causing a short between the test pins. If the pins are clean, replace the control.
*E-98	Temperature Delta Error	Temperature of the cavity sensors 1 and 2 differ by more than 3.3°C (6°F). ■ Remove product and allow the warmer to cool down. ■ Verify that the product sensor is clean and operating correctly. ■ Set the power switch to the OFF position to clear the error code. ■ If error persists, the cavity sensor should be replaced by a qualified service technician. Contact service.
E-99	Hardware Over Temp	■ Inspect the connections and condition of high limit bimetal thermostat. ■ If applicable, make sure that the compartment fan motor is operating. ■ Contact service if error persists.

Code	Refers to	Action required
*EFAn	Fan or Fan Sensor Failure	■ Check to make sure the fan sensor wires did not disconnect from the fan or the control board. ■ If the error persists after checking the wires, replace the fan.
Flashing yellow iv fluid temperature status indicator	Flashing yellow LED is illuminated within iv storage shelf	■ Set the power switch to the OFF position. ■ Remove any iv bags from inside the warmer. ■ Set the power switch to the ON position. ■ Visually check to see if flashing yellow LED remains illuminated. ■ Contact service if error persists.

*All non-critical codes can be cleared using the standby button. Critical errors (marked with a *) can only be cleared by setting the power switch at the rear of the warmer to the OFF position and allowing the warmer to cool.

How to Replace a Fuse





WARNING: Fire and electrical shock hazard.

Replace with equivalent type of fuse only. See the warmer label for fuse type information. Access should be made by qualified service technicians only.

Procedure

To replace a fuse, do the following.

Step	Action
1.	Press and hold the standby button until the screen turns off, then release the button.
2.	Set the power switch ① to the OFF (0) position.
3.	Unplug the power cord from the electric power source and the power inlet on the warmer.
4.	Pinch the side tabs and pull out the fuse drawer ②.
5.	Remove the fuse ③.
6.	Replace the fuse with a new fuse.
7.	Re-install the fuse drawer.

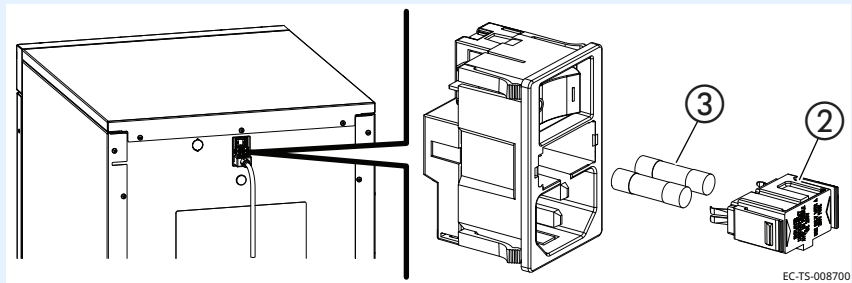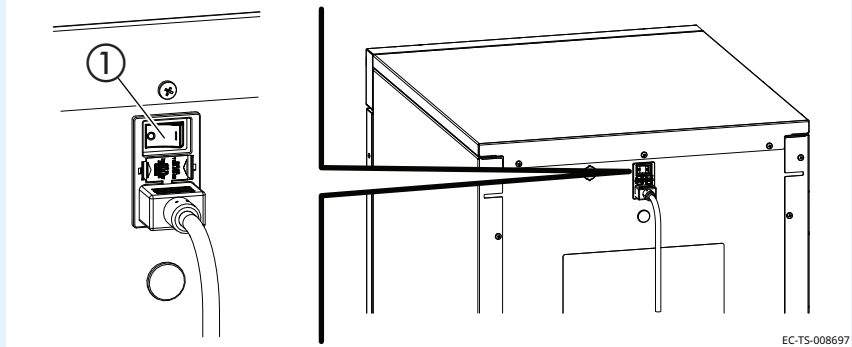

Result

The fuse has now been replaced.

This page intentionally left blank.

Guidance and Manufacturer's Declaration

The warmer requires special precautions regarding EMC (Electromagnetic Compatibility) and needs to be installed and put into service according to the EMC information provided in the accompanying documents.

Portable and mobile RF communications equipment can affect medical electrical equipment.

A risk of increased emissions or decreased immunity may result if the power cord is altered or a manufacturer supplied power cord is not used.

The warmer should not be used adjacent to or stacked with other equipment.

The essential performance of the warmer is to not exceed an internal temperature of 180°F (82°C) for blanket warmers or 150°F (66°C) for fluid warmers.

Electromagnetic emissions

The warmer is intended for use in the electromagnetic environment specified below.

Emission test	Compliance	Electromagnetic environment - guidance
RF emissions; CISPR 11	Group 1	The warmer uses RF energy only for internal function. Therefore, its RF emissions are very low and are not likely to cause any interference in nearby electronic equipment.
RF emissions; CISPR 11	Class A	The warmer is suitable for use in all establishments, other than domestic establishments and those directly connected to the public low-voltage power supply network that supplies buildings used for domestic purposes.
Harmonic emissions; IEC 61000-3-2	Class A	
Voltage fluctuations/Flicker emissions; IEC 61000-3-3	Complies without conditions	

Continued on next page

Continued from previous page

Electromagnetic immunity

The warmer is intended for use in the electromagnetic environment specified below.

Immunity test	IEC 60601 test level	Compliance level	Electromagnetic environment - guidance
Electromagnetic discharge (ESD) IEC 61000-4-2	±8 kV contact ±8 kV air	±8 kV contact ±8 kV air	Floors should be wood, concrete or ceramic tile. If floors are covered with synthetic material, the relative humidity should be at least 30%.
Electrical fast transient/burst IEC 61000-4-4	±2 kV for power supply lines; ±1 kV for input/output lines	+2 kV for power supply lines	Main power quality should be that of a typical commercial or hospital environment. The warmer does not have any input/output lines.
Surge IEC 61000-4-5	±1 kV differential mode; ±2 kV common mode	±1 kV differential mode; ±2 kV common mode	Mains power quality should be that of a typical commercial or hospital environment.
Voltage dips, short interruptions and voltage variations on power supply input lines IEC 61000-4-11	<5% UT (>95% dip in UT) for 0.5 cycle 40% UT (60% dip in UT) for 5 cycles 70% UT (30% dip in UT) for 25 cycles <5% UT (>95% dip in UT) for 5 sec	<5% UT (>95% dip in UT) for 0.5 cycle 40% UT (60% dip in UT) for 5 cycles 70% UT (30% dip in UT) for 25 cycles <5% UT (>95% dip in UT) for 5 sec	Mains power quality should be that of a typical commercial or hospital environment. If the user of the warmer requires continued operation during power mains interruptions, it is recommended that the warmer be powered from an uninterrupted power supply or a battery.
Power frequency (50/60 Hz) magnetic field IEC 61000-4-8	3 A/m	3 A/m	Power frequency magnetic fields should be at levels characteristic of a typical location in a typical commercial or hospital environment.

NOTE: UT is the a.c. mains voltage prior to application of the test level.

Continued on next page

Continued from previous page

Electromagnetic emissions

The warmer is intended for use in the electromagnetic environment specified below.

Immunity test	IEC 60601 test level	Compliance level	Electromagnetic environment - guidance
Conducted RF IEC 61000-4-6 Radiated RF IEC 61000-4-3	3 V/m 150 kHz to 80 MHz 3 V/m 80 MHz to 2.5 GHz	3 V/m	Portable and mobile RF communications equipment should be used no closer to any part of the warmer, including cables, than the recommended separation distance calculated from the equation applicable to the frequency of the transmitter. Recommended separation distance $d = [3.5/3] \sqrt{P}$ $d = [3.5/3] \sqrt{P} \text{ 80 MHz to 800 MHz}$ $d = [7/3] \sqrt{P} \text{ 800 MHz to 2.5 GHz}$ where P is the maximum output power rating of the transmitter in watts (W) according to the transmitter manufacturer and d is the recommended separation distance in meters (m). Field strengths from fixed RF transmitters, as determined by an electromagnetic site survey¹, should be less than the compliance level in each frequency range². Interference may occur in the vicinity of equipment marked with the following symbol:

NOTE: At 80 MHz and 800 MHz, the higher frequency range applies. These guidelines may not apply in all situations. Electromagnetic propagation is affected by absorption and reflection from structures, objects and people.

1. Field strengths from fixed transmitters, such as base stations for radio (cellular/cordless) telephones and land mobile radios, amateur radio, AM and FM radio broadcast and TV broadcast cannot be predicted theoretically with accuracy. To assess the electromagnetic environment due to fixed RF transmitters, an electromagnetic site survey should be considered. If the measured field strength in the location in which the warmer is used exceeds the applicable RF compliance level above, the warmer should be observed to verify normal operation. If abnormal performance is observed, additional measures may be necessary, such as reorienting or relocating the warmer.

2. Over the frequency range 150 kHz to 80 MHz, field strengths should be less than [V] V/m.

Continued on next page

Continued from previous page

**Electromagnetic
immunity distance**

The warmer is intended for use in an electromagnetic environment in which radiated RF disturbances are controlled. The user of the warmer can help prevent electromagnetic interference by maintaining a minimum distance between portable and mobile RF communications equipment (transmitters) and the warmer as recommended below, according to the maximum output power of the communications equipment.

Rated maximum output power of transmitter W	Separation distance according to frequency of transmitter m		
	150 kHz to 80 MHz $d = \left[\frac{3.5}{3} \right] \sqrt{P}$	80 MHz to 800 MHz $d = \left[\frac{3.5}{3} \right] \sqrt{P}$	800 MHz to 2.5 GHz $d = \left[\frac{7}{3} \right] \sqrt{P}$
0.01	0.117	0.117	0.233
0.1	0.369	0.369	0.738
1	1.167	1.167	2.333
10	3.689	3.389	7.379
100	11.667	11.667	23.333

For transmitters rated at a maximum output power not listed above, the recommended separation distance d in meters (m) can be estimated using the equation applicable to the frequency of the transmitter, where P is the maximum output rating of the transmitter in watts (W) according to the transmitter manufacturer.

NOTE: At 80 MHz and 800 MHz, the separation distance for the higher frequency range applies. These guidelines may not apply in all situations. Electromagnetic propagation is affected by absorption and reflection from structures, objects and people.

Warranty

Introduction

Pedigo Products, Inc. warrants to the original purchaser that any original part that is found to be defective in material or workmanship will, at our option, subject to provisions hereinafter stated, be replaced with a new or rebuilt part.

Warranty Period

The original parts warranty period is as follows:

- The labor warranty remains in effect one (1) year from installation or fifteen (15) months from the shipping date, whichever occurs first.
- The original parts warranty remains in effect three (3) years from installation of appliance or thirty-nine (39) months from the shipping date, whichever occurs first.

Exclusions

This warranty does not apply to:

- Calibration
- Equipment damage caused by accident, shipping, improper installation or alteration.
- Equipment used under conditions of abuse, misuse, carelessness or abnormal conditions including equipment subjected to harsh or inappropriate chemicals including but not limited to compounds containing chloride or quaternary salts, poor water quality, or equipment with missing or altered serial numbers.
- Any losses or damage resulting from malfunction, including loss of contents or consequential or incidental damages of any kind.
- Equipment modified in any manner from original model, substitution of parts other than factory authorized parts, removal of any parts including legs, or addition of any parts.
- Collateral or incidental damage as a direct result of servicing equipment built into a wall structure is not covered under warranty. It is the responsibility of the owner to bear all expense related to structural repairs including, but not limited to, external electrical connections and wiring, and the removal or replacement of caulk, grout, tile, or wall covering of any kind. A service access panel for built-in equipment installations is strongly recommended.

Conclusion

This warranty is exclusive and is in lieu of all other warranties, expressed or implied, including the implied warranties of merchantability and fitness for purpose. In no event shall the company be liable for loss of use, loss of revenue, or loss of contents or revenue, or for indirect or consequential damages. This warranty is in lieu of all other warranties expressed or implied and Pedigo Products, Inc. neither assumes or authorizes any persons to assume for it any other obligation or liability in connection with Pedigo Products, Inc. equipment.



Printed in the U.S.A.

Specifications are subject to change without notice.

Pedigo Products, Inc.

4000 S.E. Columbia Way

Vancouver, WA 98661

Phone: 800.822.3501 | Fax: 360.696.1700

www.pedigo-usa.com

Made in the U.S.A